explorē 12

User Guide

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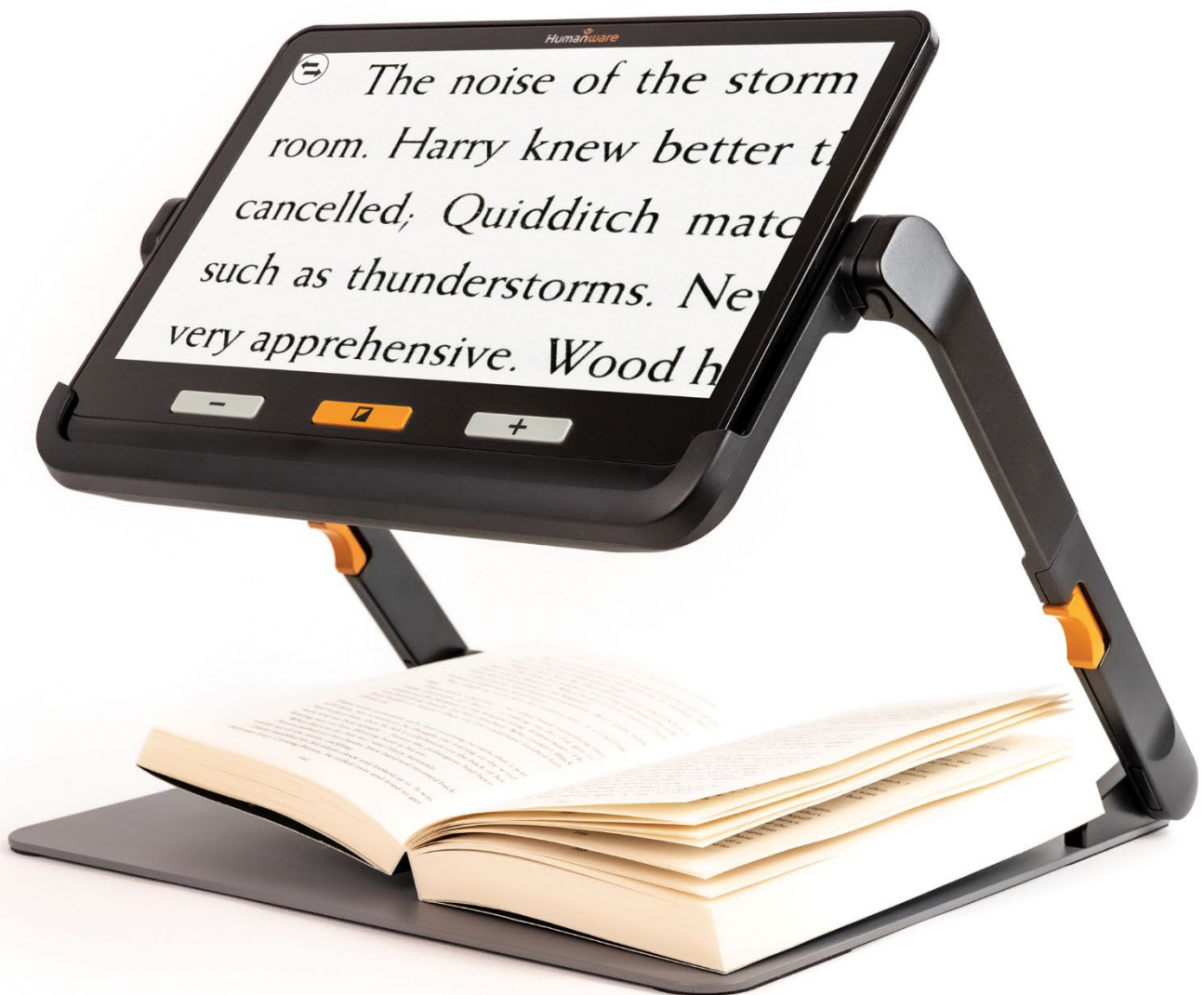
TABLE OF CONTENTS

Chapter 1 – Overview.....	3
Introduction.....	3
Box Contents.....	4
Physical Description.....	5
Getting Started.....	7
Chapter 2 – Using Your System.....	10
Powering On.....	10
Powering Off.....	10
Magnifier Mode.....	11
Zooming In and Out.....	11
Horizontal panning of a live image (folding stand only).....	12
Switching Cameras.....	13
Changing Contrast.....	13
Turning lights On and Off.....	14
Lines and blinds.....	14
Autofocus.....	15
Freezing an Image.....	16
Navigating a Frozen Image.....	16
Gallery Mode.....	16
Accessing the Gallery.....	17
Deleting Images from Gallery.....	17
Using an External Screen.....	18
Chapter 3 – Exporting Images to A Computer.....	18
Chapter 4 – Menu.....	18
Accessing the Menu and Changing Settings.....	18
Chapter 5 – Updating the System.....	22
Chapter 6 – Recharging the System.....	23
Chapter 7 – Troubleshooting.....	24
Appendix A – Safety and Maintenance.....	27
Appendix B – Specs.....	29
Appendix C – FCC Info.....	30
Appendix D – Warranty.....	30

CHAPTER 1 – OVERVIEW

INTRODUCTION

Thank you for purchasing the HumanWare **explorē 12**, the user-friendly, portable, electronic handheld magnifier with a large, easy-to-read twelve-inch screen.

explorē 12 includes essential and easy-to-use tools to address low vision or vision loss needs, regardless of your age. With superior magnification, this tablet allows you to continue daily activities like reading and viewing photos or maps and maintain productivity in your work or classroom environment.

When used with its foldable stand, **explorē 12** rises above the table, allowing for plenty of writing room. The touchscreen's zoom-in and pan functions also allow you to read without moving your documents. In addition, its lightweight design makes it an ideal companion for on-the-go use, facilitating your active and mobile lifestyle.

At HumanWare, we know that in addition to the best available image quality, a good magnifier needs to be portable, ergonomic and uncomplicated so that it can be used immediately. The **explorē 12** is a carefully considered and meticulously designed product that will capably meet your needs and expectations.

Carefully read the operating and safety instructions described in this user guide before using the device. If you have any questions about your system, please contact HumanWare Customer Service (contact information is located at the back of this user guide). Please note, software updates may have occurred since the printing of this guide. To obtain the most recent version, please visit the Support page at www.humanware.com


Copyright 2022 by Technologies HumanWare Inc., all rights reserved. No part of this publication may be reproduced or distributed in any form or by any means or stored in a database or retrieval system without the prior written permission of Technologies HumanWare Inc.

BOX CONTENTS


- **explorē 12** electronic magnifier
- Folding and transportable reading stand
- International power adapters
- Type-C USB and high-resolution video cables
- User and Getting started manuals
- Neoprene carry case, screen protector, cleaning cloth and shoulder bag

PHYSICAL DESCRIPTION


Front face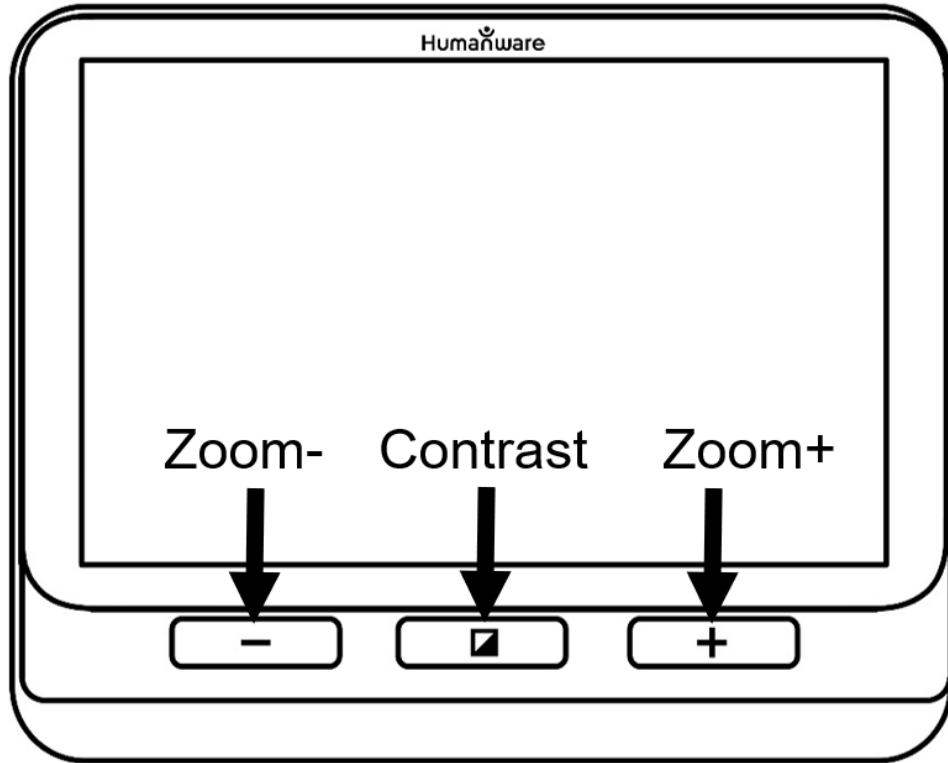


Left side

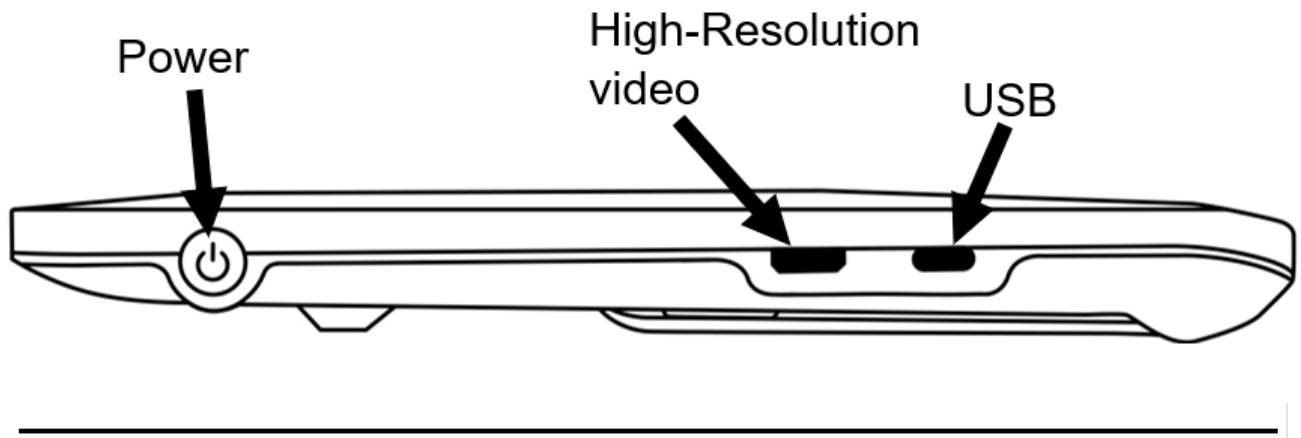Top edge

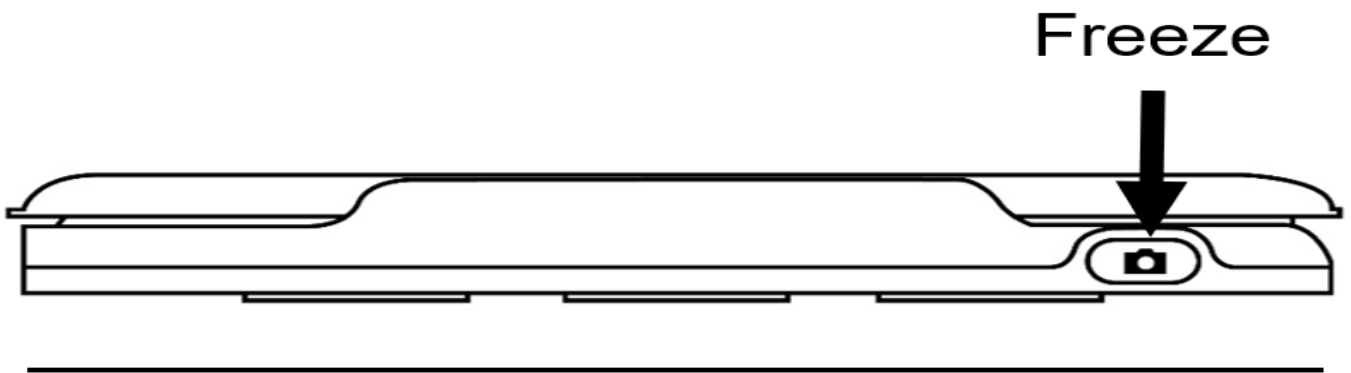
Backside

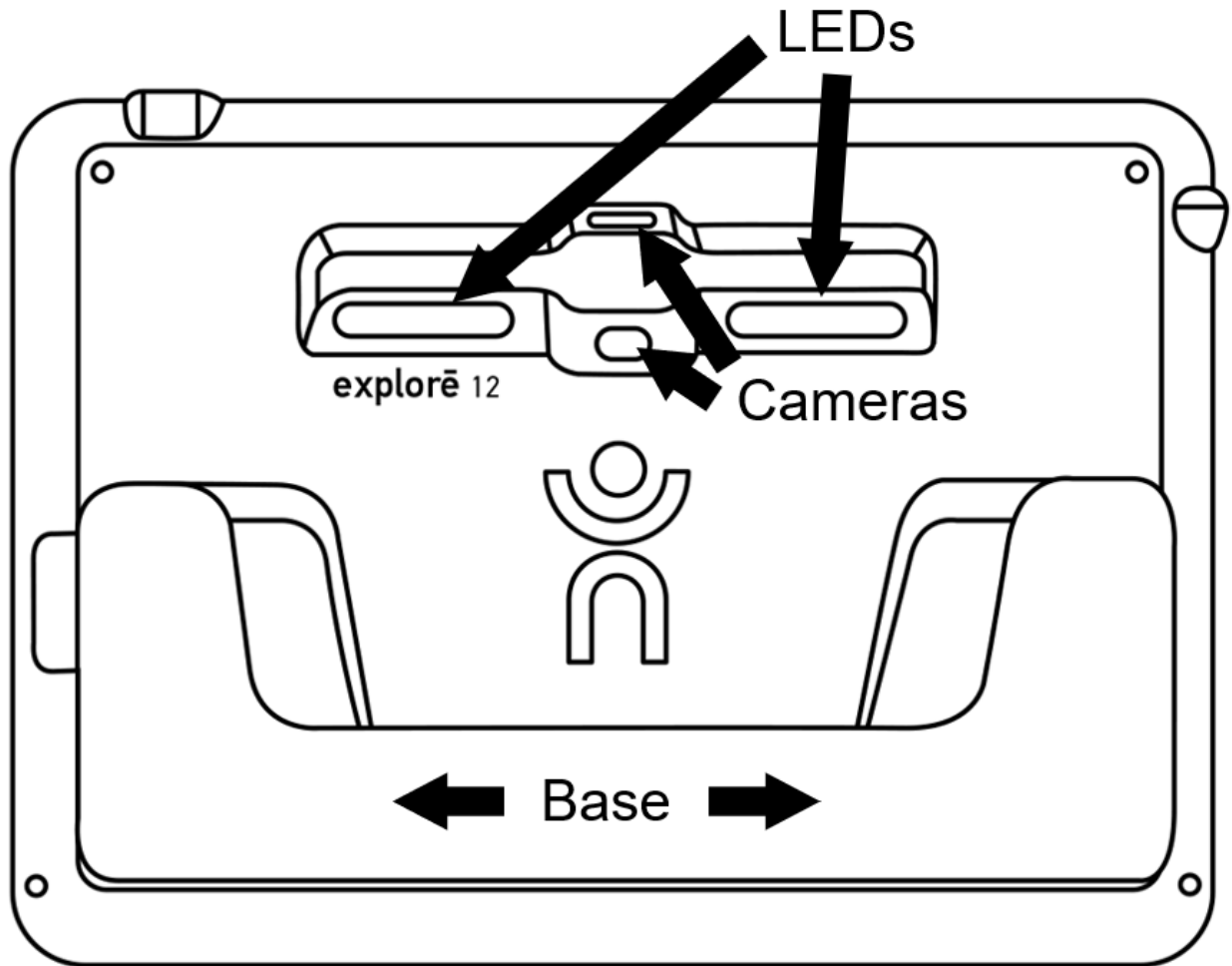
GETTING STARTED

1. Connect the USB cable to the USB charger.
2. Hold the explorē 12 so that the screen side is facing you. **Carefully** connect the cable to the USB port located on the left side of your device and plug the charger into an outlet. A battery icon (charge level indicator) appears for 3 seconds at the top, right of the screen.
3. For best performance, recharge until the battery is full. Press the **Power** button to obtain a charge level reading. The device will warm up during the charging process – this is normal.



4. Once the device is fully charged, carefully disconnect the cable.
5. Press the **Power** button or unfold the reading base to begin using the device.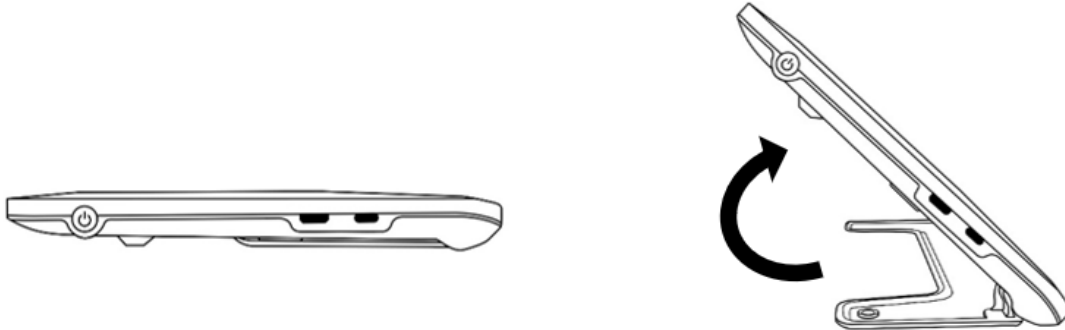

6. You can also unfold the (optional) reading stand and dock the device in the tray to use it in "Stand Mode."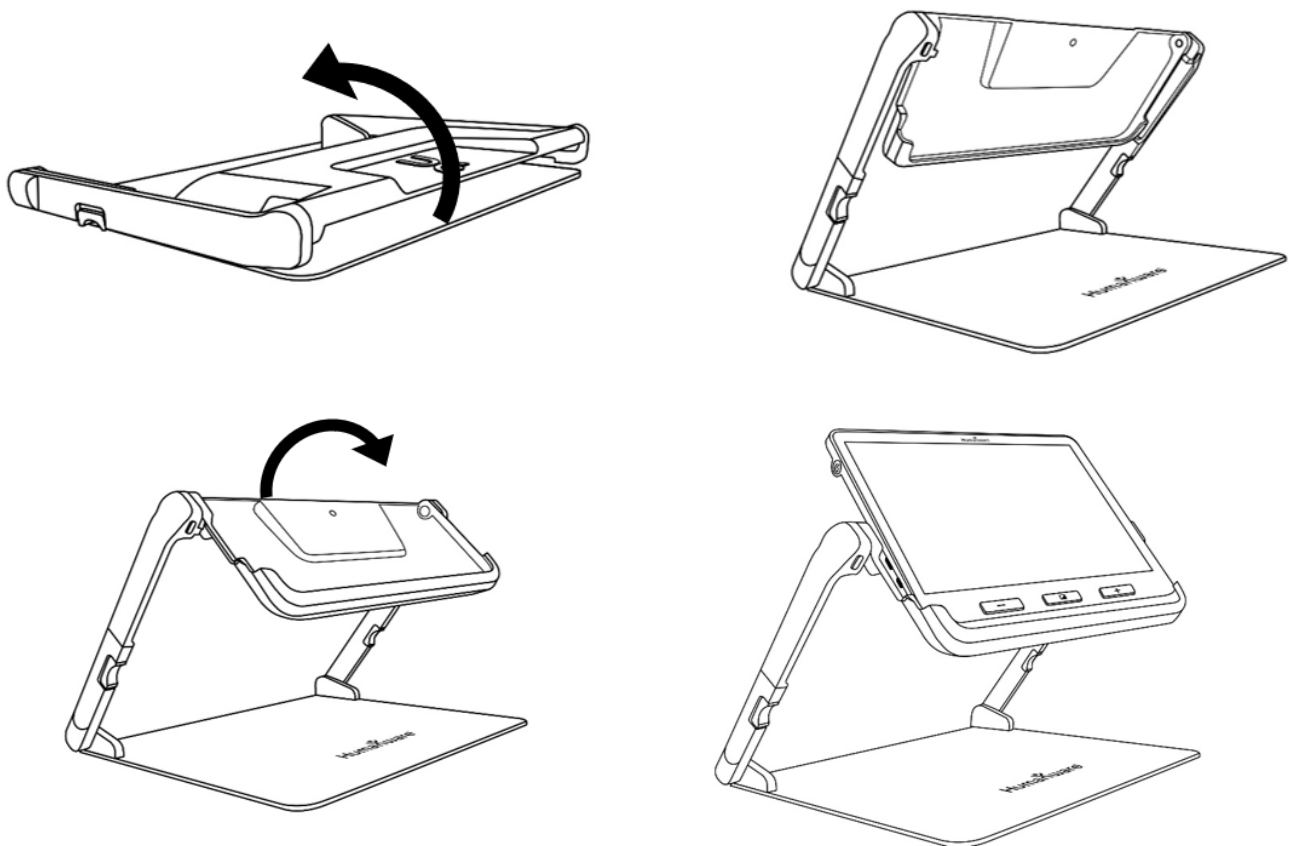

7. When you are finished working in "Stand Mode," slide the device out from the tray and press the two **orange buttons** on each side of the stand to collapse the unit.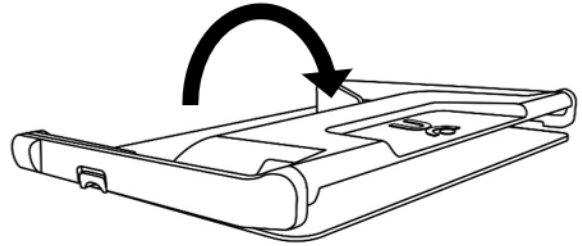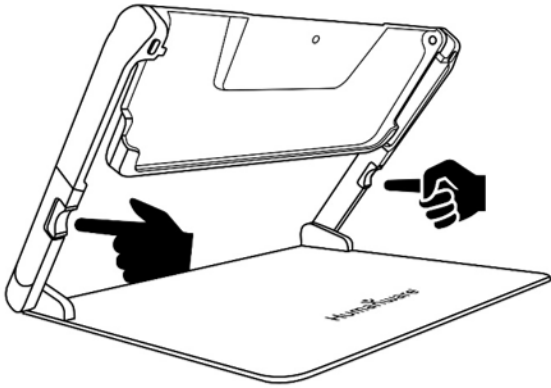


CHAPTER 2 – USING YOUR SYSTEM

POWERING ON

To start the magnifier, press the **Power** button on the left side of the device. The device can then be held by the sides, just as you would hold a tablet.

You can also turn on the device by unfolding its base or by sliding the device into the reading stand.

IMPORTANT: Some users may prefer to familiarize themselves with **device settings** before proceeding with device operation. If this is the case, please refer to Chapter 4: [Accessing the Menu and Changing Settings](#)

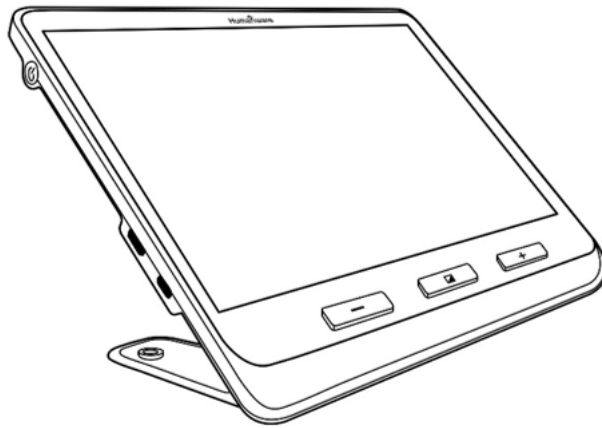After a few seconds, the explorē 12 screen will display the image captured by the near camera. This is the **Magnifier mode**.

POWERING OFF

Press the **Power** button again to turn the device Off

MAGNIFIER MODE

Magnifier mode is explorē 12's primary mode of operation. With the base opened, place the unit on a tabletop to magnify close objects or documents. With the base snapped shut (closed position), you can magnify distant objects while using the device in hand-held mode. With the device placed in the (optional) reading stand, you can magnify close objects or documents that are placed under the device. Magnification level ranges from 1x to 15x when the unit is placed on a tabletop or in the optional stand. Magnification level ranges from 2x to 30x in hand-held mode.

ZOOMING IN AND OUT

To zoom in, press or press and hold **Zoom+** until the image is the appropriate size. Similarly, you can zoom out by pressing, or pressing and holding **Zoom-** until you are satisfied with the image size.

Alternatively, you can enable the **pinch and zoom** gesture (decrease and increase the distance between two fingers) from the menu to zoom in and out of an image.

Users can also set their **preferred zoom level** for each mode (base & handheld, folding stand and distance) making it easier to move from task to task efficiently. To do this, access **Menu**, then **Preferred Zooms** and select zoom levels for any/all three modes by tapping the **right arrow** to access zoom levels. Zoom levels range from **2x** to **15x** for all three modes. Each mode also has a **Previous** option, by tapping **Previous**, the user returns to whichever zoom level was selected earlier.

The explorē 12 retains the zoom level between close-up and distance viewing when the user moves between viewing modes, allowing for a seamless transition between tasks like distance viewing and reading a document.

HORIZONTAL PANNING OF A LIVE IMAGE (FOLDING STAND ONLY)

While viewing a document or an object in magnifier mode using the foldable reading stand, you can pan left or right through the image by sliding your finger across the screen in the direction that you wish to pan.

You can slide two fingers from top to bottom across the middle of the screen, and the image will zoom to 1x, allowing you to reposition your material. The image will return to the previous zoom level once you release your fingers from the screen.

Left and right panning arrows at the bottom of the screen automatically display **for zoom levels 4x and higher**.

- Tapping and holding the panning arrows will perform an incrementally smooth pan to the left or right of the camera's viewable area.
- If you need to pan to an exact spot on an image, text or object, *short press and release* the left or right panning arrows to pan one increment at a time.
- Double-tap the panning arrows to go to the extreme end of either margin.

Note: You must use the Near camera on the stand for the Horizontal panning feature to work.

PANNING IN DISTANCE VIEWING MODE

You can horizontally pan using gestures or on-screen icons when the device is in distance viewing mode. This functionality benefits the user in locating where in the distance they need to zoom in for closer inspection. Two-finger

“Overview” gesture is available for use with the distance viewing mode. You can slide two fingers from top to bottom across the middle of the screen, and the image will zoom to 1x, allowing you to note where the zoom was effectuated. The image will return to the previous zoom level once you release your fingers from the screen.

SWITCHING CAMERAS

Icon art makes it easy for the user to recognize the functionality of the on-screen icons that switch camera modes. Tap the **Mountains** icon to access distance mode, and the **Text** icon to access document/near viewing mode.

By default, the magnifier mode is set to the Near Camera view when the base or the stand is opened, however, you can toggle to the Distance Camera (designated by the **Mountains** icon at the top, left of the screen).

Alternatively, you can toggle the camera view by selecting **Toggle Camera** in the **Settings** menu.

CHANGING CONTRAST

explorē 12 allows you to customize contrast to suit your needs by pressing the orange **Contrast** button (located at the bottom, center of the device) to change contrast levels.

Default contrast modes are: full color, black on white, white on black, black on yellow, yellow on black.

Other contrast modes are available and can be activated via the **Settings** menu. They include black on orange, orange on black, black on cyan, cyan on black, black on purple, purple on black, black on green, green on black, blue on yellow, yellow on blue, white on blue and blue on white, red on white and red on black.

TURNING LIGHTS ON AND OFF

Press and hold **Zoom+** and **Zoom-** simultaneously to turn lights On and Off.

Note: Near camera must be selected to turn lights On.

Lighting control setting for each viewing mode


LED lights can be set to **On**, **Low**, or **Off** for each viewing mode, thus enabling you to set up the device for use in various environments or for specific tasks.

Access **Lights** from the Menu, select a viewing mode (**Base** or **Folding Stand & Handheld**), then select **On**, **Low**, or **Off** for each of the viewing modes.

LINES AND BLINDS

The explorē 12 allows you to add a horizontal line (left image) or blinds (right image) over your image for easier reading.

Lorem ipsum dolor
 sit amet, consectetur
 adipiscing elit. Mae
 porttitor congue m

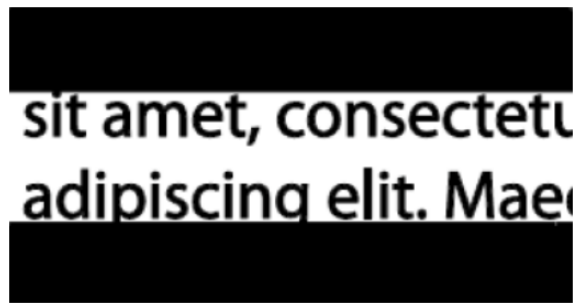sit amet, consectetur
 adipiscing elit. Mae

To add a line, press and hold **Contrast** and **Zoom+** simultaneously. Repeat to remove the line.

To add blinds, press and hold **Contrast** and **Zoom-** simultaneously. Repeat to remove the blinds.

AUTOFOCUS

explorē 12 has an autofocus feature that can be set to **locked**, **unlocked**, or **manual** from the menu.

To temporarily lock autofocus, double-tap on the screen while magnifying an image. A **Padlock** icon will appear at the top right corner of the screen. double-tap the icon again to unlock autofocus.

Note: Changing the zoom level of the image will automatically unlock autofocus.

When autofocus is set to **unlocked** (system default), explorē 12 will dynamically adjust its focus to the object currently being viewed. Focus on a specific area can be engaged and refined with a quick touch anywhere on the screen.

When autofocus is set to **locked**, or when the magnifier is used in base mode, explorē 12 will maintain its current focus regardless of any movements or changes.

When autofocus is set to **manual**, touch any area on the screen to focus on that specific location. Note that the **manual** setting only works in distance viewing mode and stand mode. Base mode remains locked when **manual** mode is selected.

FREEZING AN IMAGE

explorē 12 enables you to freeze an image (essentially taking a snapshot) while in magnifier mode for easier reading. This is a handy feature when reading product labels on stores' top shelves. To freeze an image, press the **Freeze** button while aiming the magnifier toward what you need to read. The snapshot will automatically display on your screen until you return to Magnifier mode.

To return to magnifier mode, press the **Freeze** button once again.

NAVIGATING A FROZEN IMAGE

You can zoom in on still images by pressing or pressing and holding **Zoom+** until the image is the appropriate size. Zoom out by pressing or pressing and holding **Zoom-** until you are satisfied with the image size.

Alternatively, you can enable the **pinch and zoom** gesture (decrease and increase the distance between two fingers) from the menu to zoom in and out of a frozen image. Hold your finger down and slide it across the screen to see other parts of a frozen image.

As with a live image, you can press **Contrast** to change contrast until you find one that is comfortable.

GALLERY MODE

To save a live or frozen image to your Gallery, press and hold **Freeze** for a half-second while in Magnifier mode. An hourglass **Saving** icon will briefly display on the screen. The image is now saved in the Gallery.

Alternatively, you can enable the **tap-and-hold** gesture (keep your finger down on the screen) from the menu to save an image in the Gallery.

ACCESSING THE GALLERY

To access images in the Gallery, press **Contrast** and **Freeze** simultaneously. To select an image, swipe your finger left or right on the screen. You can also press the arrows on the screen to choose an image. Images are added sequentially to the gallery and an image number along with total number of captured images (example: 3/17) will briefly appear at the bottom, right of the screen when carrying out either of the two operations outlined above.

You can zoom in and out of saved images by pressing or pressing and holding **Zoom+** and **Zoom-**. Alternatively, you can enable the **pinch and zoom** gesture (decrease and increase the distance between two fingers) from the menu to zoom in and out of a saved image.

To exit the Gallery and return to Magnifier mode, press the **Back** icon or press and hold **Contrast**.

DELETING IMAGES FROM GALLERY

To delete an image from the Gallery, select the image that you wish to delete, then press and hold **Freeze**. A “Delete/Cancel” pop-up will appear, tap one of the two options.

You can also **tap and hold** your finger on the screen to delete an image. A “Delete/Cancel” pop-up will appear, tap one of the two options.

You can delete all of the images from the gallery by selecting **Delete Gallery** from the menu, then clicking **Yes**. Click **No** to cancel deletion.

Note: When a user enters a submenu and returns to the magnifier, the device remembers their last viewing mode. For example, when you look at an object in the distance, capture an image, then go into the Gallery to view the image, you will automatically return to distance viewing mode when you tap the **Back** icon, allowing you to continue your previous activity.

USING AN EXTERNAL SCREEN

explorē 12's content can be displayed on an external screen to obtain a larger image. To do so, connect the explorē 12 to the external screen using the High-Resolution video cable (included). Refer to Appendix B for a list of supported High-Resolution video formats.

CHAPTER 3 – EXPORTING IMAGES TO A COMPUTER

To export Gallery images to your computer, connect your device with the USB cable provided. A "PC connected" message will appear on the device. You can now copy and paste the images from your Gallery to your computer using your computer's file browser.

When connected to a personal computer, the device will act exactly like a USB thumb drive or an external drive. Gallery images are in the **Images folder** on the explorē 12.

IMPORTANT: When copying or deleting images from the device, *do not disconnect the USB cable* as some images may be lost in the process.

CHAPTER 4 – MENU

ACCESSING THE MENU AND CHANGING SETTINGS

Press and hold **Contrast** to enter the **Menu**. You can navigate the menu by swiping up or down with your finger on the screen, pressing the up/down arrows, or pressing **Zoom+** to scroll up and **Zoom-** to scroll down.

Tap an item to select it. To go back to the previous menu, press **Contrast** (orange tactile **button** at the bottom, center of the unit) or **Back** icon (circular, counterclockwise arrow at the top, left of **screen**).

Press **Contrast** again to return to Magnifier mode. To exit the menu, press and hold **Contrast** button or **Back** icon.

Menu sections include (an asterisk (*) denotes a default value):

- **Gallery** (all images contained in the Gallery)
- **Toggle camera:** Distant Camera*, Near Camera (Near Camera* in base or stand mode)
- **Autofocus:** Unlocked*, Locked (Locked* in base mode), Manual (Locked in base mode)
- **Screen Brightness** (Change the screen's brightness level): 1 to 9 (8*)
- **Touchscreen** (submenu, tap to enable gestures and vibrations)

Touch gestures can be enabled or disabled from the **Settings** menu. This customization allows users to utilize the gestures that they need or are most comfortable using.

- **Pinch & zoom** (decrease/increase distance between two fingers to zoom out/in)
- **Vibrations** (device vibrates when certain functions are activated/disactivated)
- **Tap & hold** (hold a finger down in Magnifier mode to save an image to Gallery)
- **Temporary focus lock** focus is locked in at a particular depth so that the autofocus feature does not interfere with text or objects that aren't in the desired focus zone.

- **Image overview** (swipe two fingers up/down the middle of the screen to zoom up to 1x)
 - **Toggle button** selects Mountain icon for distance viewing and text icon for close viewing, icons are displayed at the top left of the screen.
 - **Pan buttons** (for zoom 4x or higher on folding stand only) displays left and right arrows at the bottom of the screen to pan to the far left or right of the camera's viewable area while magnifying a live image
- **Auto-off** (select how long the device remains on): 1 minute, 2 minutes, 5 minutes*, 10 minutes, Never.
 - **Preferred Zoom** there are three modes to set preferred zoom. Users can select the default zoom level when turning on the device, Previous (zoom level at the time when the device was turned off), 2x* to 30x. 2x* to 15x in Stand mode with Near camera. Note that you must restart the device (power off, then back on) for any changes to take effect.
 - **Base and portable** (change default zoom level in base or portable modes)
 - **Support** (change default zoom level in support mode)
 - **Distance** (change default zoom level in distance mode)
 - **Contrast** (submenu)

A variety of contrast color combinations are included to support users who find these combinations most beneficial to their viewing needs.

- **Preferred Contrast** (start-up contrast): Previous* (last used contrast), Color, or choice of enabled contrasts
- **Menu** (change menu colors): black on white*, white on black, black on yellow, yellow on black, black on orange, orange on black, black on cyan, cyan on black, black on purple, purple on black, black on green, green on black, blue on yellow, yellow on blue, white on blue, blue on white

- **Enabled contrasts:** black on white*, white on black*, black on yellow*
- The following contrast colors *must be enabled* by the user by selecting **Menu**, then **Contrast**, then **Enabled Contrasts**, then selecting from the Contrasts list.

yellow on black*, black on orange, orange on black, black on cyan, cyan on black, black on purple, purple on black, black on green, green on black, blue on yellow, yellow on blue, white on blue, blue on white, red on black, red on white

- **Lines and blinds:** Line, Blinds, None*
- **Lights:** On*, Low, Off
 - **Base** (Base mode, On*)
 - **Foldable Stand and Portable** (Support mode, On*)
- **Menu Text Size:** Small, Medium*, Large
- **Language:** English US*, English UK, French (CA), French (FR), Spanish (US), Spanish (ES), German, Dutch, Danish, Polish, Portuguese (PT), Portuguese (BR), Norwegian, Italian, Swedish, Finnish, Russian, Czech, Japanese, Korean, Icelandic, Estonian, Turkish, Lithuanian, Hungarian, Latvian, Arabic, Chinese (Simplified), Chinese (Traditional), Hebrew, Georgian, Farsi, Kurdish
- **Display:** 50Hz, 60Hz*
- **Delete Gallery:** Clears all images from the device.
- **Factory Defaults:** Clears all personal settings on the device and resets device to factory defaults.
- **About** (View information about your device): Set date and time, Software, Hardware, Serial Number.

Users can select the date and time for the device using touch gestures or physical buttons.

Date and time can be set by accessing the **About Menu**, then **Date and time**, then tapping **year/month/day/hours/minutes** and selecting the correct date and time by tapping the **up/down arrows**. The user can also adjust the **year/month/day/hours/minutes** selection by tapping the **+** or **-** keys then pressing the **capture** button to select the value to change, arrows will appear on the screen, tap the **+** or **-** to change the value. Finally, press the **capture** button to deselect the edit field.

CHAPTER 5 – UPDATING THE SYSTEM

Step 1: Click this link or copy and paste it into your browser:
<http://support.humanware.com/en-canada/support/explore>

Step 2: Select **Download** for the latest version of the explorē 12 software.

Step 3: Connect the explorē 12 to your computer using the USB cable (provided).

Step 4: Access your computer's file explorer, then copy and paste or drag and drop your downloaded file(s) directly into your explorē 12 root directory.

Step 5: When the transfer has completed, unplug your USB cable from your computer and **restart your explorē 12**. Your device will have automatically updated.


CHAPTER 6 – RECHARGING THE SYSTEM

IMPORTANT: The battery becomes warm during recharging, this is normal. Do not recharge the battery near a heat source.

To recharge the battery using a household power outlet:

- Connect the USB cable to explorē 12's USB charger. Then, carefully connect the USB cable to your device and plug the charger into an outlet.
- Let your device charge until the battery is full. It takes approximately 3.5 hours to recharge a device with a depleted battery.

The **Battery** icon below will display momentarily when the battery starts charging or when the device is turned on during the recharge process.

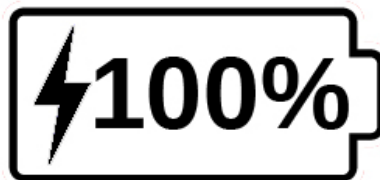The lightning bolt indicates that your device is currently being charged. The number to the right represents the percentage of battery charge remaining.

Note: The percentage of remaining battery is shown as you move between tabletop, stand and distance viewing modes, this way, you can maintain awareness of adequate charging levels to complete your tasks.

Due to lower voltage, charging from the computer USB may be slower. In some cases, your device may not charge at all, depending on the amount of power supplied by the computer. **The most effective way to recharge the device is to use the provided USB cable connected to the USB charger and recharging from a household power outlet.**

Although it may lengthen charging time, the device can be used when it is recharging.

Note: If the battery is fully discharged or hasn't been used for a very long period, there will be a delay of up to a few minutes before the device shows any activity. This is normal.

Also, your explorē 12 will become warm when in use or when being charged like all electronic devices. This is normal.

Battery safety precautions

- Do not disassemble, dismantle or modify the battery.
- Use only the specified HumanWare charger.

IMPORTANT: There is a risk of overheating, fire, or explosion if the battery is put in a fire, heated, subjected to impact, placed in contact with water, or if its terminals are shorted.

CHAPTER 7 – TROUBLESHOOTING

explorē 12 does not power On

Press and hold **Power**. If the unit still does not turn on, your device's battery may require recharging.

Connect the provided USB cable with the USB charger into a power outlet or just the USB cable into a running computer. If the battery was completely discharged, it might take a few minutes before the device shows any signs of activity.

Screen remains black

The image may be black if the unit is lying on a table or any flat surface. Lift your magnifier to see whether this is the reason why your screen is black. If this does not solve the problem, restart the device.

Image is out of focus in magnifier

Make sure Autofocus is set to **Unlocked** in the menu, then move the device back and forth over the document you are viewing. This will force explorē 12 re-focus on the document.

explorē 12 shuts down after inactivity

This is normal behavior that is intended to preserve the battery life. By default, explorē 12 will shut down after 5 minutes of inactivity. You can configure the default auto-off timer to 1 minute, 2 minutes, 5 minutes, 10 minutes or Never.

Colors are difficult to detect

Be certain that there is sufficient ambient light around the device.

Choose a different color combination by pressing the **Contrast button** while in magnifier mode.

Screen glare

Certain objects and reading materials are made with reflective surfaces. Common examples include compact disc jewel cases, varnished book covers, varnished prints, ID cards / credit cards with glossy surfaces, mobile phone screens (Gorilla glass) and plasticized items such as drivers' licences.

When using these materials, tilt them in different directions under the reader to reduce glare. Another option, whenever possible, is turning off the reader lights by pressing and holding **Zoom+** and **Zoom-** simultaneously.

Another option is to try different contrast modes by clicking the orange **Contrast** button at the bottom, center of the device.

explorē 12 does not respond

Be sure that the explorē 12 is not connected to power via a USB cable, if it is, unplug it and then press and hold **Power** for 10 seconds. The system will shut down. Then, press **Power** for two seconds to turn explorē 12 back on.

Other issues

To access **Factory Defaults** in the menu, press and hold **Contrast**, scroll down to **Factory Defaults** and select **Yes**.

APPENDIX A – SAFETY AND MAINTENANCE

You must **NOT** stare at the LED light source located on the backside of the magnifier.

Power **Off** your explorē 12 before cleaning it.

Use only the cables provided with your explorē 12.

Battery

The explorē 12 has an internal rechargeable Lithium-Ion battery. A qualified HumanWare technician must replace the battery.

- Do not use or leave the unit near a heat source or fire. Do not store at high temperatures.
- Only use USB-compliant power supplies to charge the battery.
- Do not disassemble or modify the unit.
- Keep away from water or high humidity environments. Do not submerge in any liquid or allow the device to get wet.
- Do not strike or throw the unit.
- Do not pierce or strike the unit with sharp or blunt objects.
- If the battery leaks and its liquid gets into the eyes, do not rub the eyes. Instead, rinse eyes with clean running water and seek immediate medical attention to prevent injury.
- If the battery leaks and its liquid comes into contact with skin, immediately rinse the area with clean running water to prevent injury.

Cleaning the LCD screen

Be sure the device is turned off and unplugged.

Use only microfiber cloth (included) to clean the LCD screen. Be sure to wipe the LCD screen **gently**.

If you choose to clean your LCD screen with a liquid cleaner, be sure the cleaner is appropriate for LCD screens. Distilled water, isopropyl alcohol and vinegar-water solutions (no more than 10% vinegar) are safe cleaning liquids for your LCD screen.

Use only a minimal amount of liquid to prevent it from seeping into the device. Be sure that liquids do not remain on the screen. Repeat the previous steps for spots that are hard to remove.

Never use household cleaners to clean your explorē 12.

Never spray any liquid directly on your LCD screen.

Never use cleaning products that contain ethyl alcohol, ethyl acid, ammonia, acetone, or methyl chloride. These products will damage the screen and possibly the internal components.

Never use products that contain isopropyl alcohol to clean the lenses of the cameras.

Never use a paper towel to clean your LCD screen. This may produce scratches.

APPENDIX B – SPECS

Tablet size: 282.5 x 202 x 24 mm

Stand size: 345 x 270 x 35.5 mm

Display: 12" ISP LCD

Weight: 1134 g

Weight with stand: 2268 g

Power Input: USB Type C connector. 5V@3A, 9V@2A or 12V@1.5A

Battery life: 3.5 hours base mode and/or distance mode and 2 hours in stand mode with LEDs on

Charge Time: 4 hours (when device is off)

High-Resolution video output Compatibility:

Supported High-Resolution video output formats:

1280x720p 60Hz, 50Hz

1920x1080p 60Hz, 50Hz, 24Hz

1920x1080i 60Hz, 50Hz

No support for DVI emulation.

NOTE: Hi-resolution performance may vary from specifications with the use of some external screens or monitors.

Operating conditions:

Temperature: 10 °C to 40 °C / 50 °F to 104 °F

Humidity: 20% to 80%

Transport and storage conditions:

Temperature: -20 °C to 60 °C / -4 °F to 140 °F

Humidity: 10% to 95%

APPENDIX C – FCC INFO

FCC Statement:

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC Warning:

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Note: This unit was tested with shielded cables on the peripheral devices. **Shielded cables must be used with the unit to ensure compliance.**

APPENDIX D – WARRANTY

HumanWare warrants explorē 12, effective from the date of purchase, to be free of any defects in material and workmanship for a period of two (2) years unless otherwise required by law in the country or region of purchase.

This warranty is non-transferable and applies to all cases where the damage is not a result of improper use, mistreatment, or negligence. Improper use is the use of this device other than described in this manual. In no event shall HumanWare or its distributors be liable for indirect or consequential damages.

No replacement or repair covered by the warranty will be carried out unless the system is accompanied by a copy of the original bill of purchase. Please keep your original receipt. If the system must be returned, please use the original packaging.

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